CONNECT WITH US!

facebook.com/cambridgemaskco

cambridgemask

twitter.com/cambridgemaskco

info@cambridgemask.com

WWW.CAMBRIDGEMASK.COM

CAMBRIDGE MASK CO

PRO

USER MANUAL

CAMBRIDGE MASK CO

WWW.CAMBRIDGEMASK.COM

CE 0493

EN149:2001 + A1:2009 FFP2 R D 

 Cambridge Environmental Solutions Ltd, RM 1001-2 10/F Wilson House, 19-27 Wyndham Street Central Hong Kong, EU Importer Cambridge Mask Company, Unit 179, 23 King Street, Cambridge, CB1 1AH, UK
Conformity assessment to EU PPE Regulation, (EU) 2016/425, carried out by Centexbel, Technologiepark 70, 9052 Gent-Zwijnaarde, Belgium.

Store in a cool, dry place. Hand wash with warm water and washing soda. Hang dry.
Washing process will only clean the exterior surfaces and will not clean the filter layers.

CONTENTS

02 English	04 Spanish	06 German
08 French	10 Chinese	12 Korean
14 Japanese	16 Hindi	18 Arabic
20 Thai	22 Vietnamese	24 Indonesian
26 Finnish	28 Polish	

THANK YOU


Thank you for choosing Cambridge Mask. We work hard to make sure you and those you care about can get outside and enjoy life without worrying about the air you breathe or the way you look.

If you have any feedback about our masks, please get in touch via info@cambridgemask.com

YOUR CAMBRIDGE MASK

We are proud of the high quality product that we make. As a non-disposable half-face respirator, Cambridge Mask can be worn far longer than most disposable industrial masks. Once the filters are "full" of particles, the mask needs to be changed. Our PRO masks are non-disposable, meaning that you can use them for a maximum of 340 hours or 3-6 months. The length of time they are suitable for depends on how polluted the environment is when you use it. Replace your mask once you feel the breathing resistance increasing significantly from your normal use.

CONTENTS: Mask, Instructions, Headstrap, Adhesive Silicone Nose Foam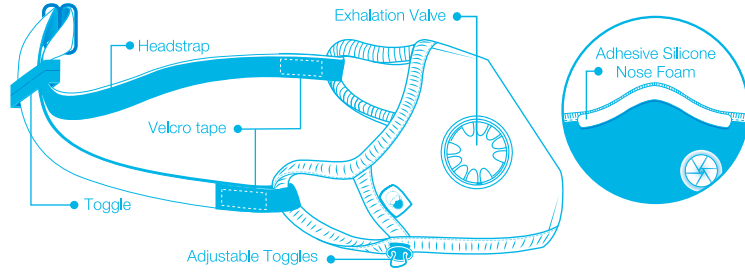

FITTING YOUR CAMBRIDGE MASK

1. The ear straps are designed to fit behind the ears to make the mask comfortable to wear. Please make sure there is a tight seal. Press the mask to your face when you put it on to help form a seal.
2. When you wear the mask be sure to adjust the noseclip and toggles so that there is a tight seal around **all** of the face. This ensures that the air is flowing through the mask and not around it.
3. Included is a headstrap. If required, affix the headstrap as indicated in the diagram above using the velcro. Then tighten the headstrap as necessary using the toggle at the back.
4. Included is an adhesive silicone nose foam. This is intended for wearers who are finding their glasses are fogging while using the mask. Fix to the inside of the mask, under the noseclip, if required.


SIZING

Measure your face using the instructions below to select your size.

A. Measure from the bridge of your nose to 20mm under your chin.

B. Measure cheekbone to cheekbone for the width of the mask.

Size	XS	S	M	L	XL
A (mm/inch)	120mm /4.7inch	140mm /5.5inch	150mm /5.9inch	170mm /6.7inch	175mm /6.8inch
B (mm/inch)	135mm /5.3inch	170mm /6.7inch	190mm /7.5inch	200mm /7.8inch	220mm /8.6inch
Weight	11-18kg 25-40lbs	20-45kg 42-100lbs	50-78kg 110-170lbs	80-90kg 175-200lbs	95kg+ 200lbs+

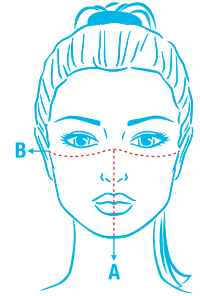TESTING

Verified by independent test laboratories. Please find the EU Declaration of Conformity using the following link.
<https://cambridgemask.com/eu-declaration-of-conformity/>

- Particulate Filtration Efficiency >99%
- Breathing resistance <2.4 mbar
- Bacterial Filtration Efficiency >99%
- Viral Filtration Efficiency >99%

STORAGE & GUIDELINES

1. Keep your mask sealed in a cool, dry place with low humidity (-20 C to +30 C).
2. The Cambridge Mask can be gently hand washed with washing soda and warm water to remove dirt from the outer fabric layer.
3. Mask must be fitted properly to filter effectively. Infants and children should be supervised when wearing a mask. Mask does not filter industrial chemicals or oil-based particles such as paint. Do not exceed recommended maximum hours of usage.
4. The mask is intended to provide protection of the wearer's respiratory system from airborne substances and mixtures which are hazardous to health. The mask is intended as a lifestyle product for use by the general public. It is not intended for professional, industrial or other occupations.

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